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Substances	Concentration ng/ml
11-nor- D9-THC-9 COOH (THC)	50
Cannabinol	20 000
11-nor- D8-THC-9 COOH	30
D8-THC	15 000
D9-THC	15 000

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## TEST FOR THE DETECTION OF THC AND ITS RESIDUES THE URINE

The EXACTO® CANNABIS test is a rapid immunochromatographic test allowing the detection of detect the active substance (THC) and its metabolites in urine.

### CANNABIS

- Origin**  
Mixture of leaves and flower tops from the cannabis plant
- Properties**  
Psychoactive action
- Effects**  
Exaltation, ecstasy, sleep, mental disorders, loss of intellectual faculties
- Duration of detection in urine**  
1 to 3 days for isolated consumption,  
8 to 12 days for regular consumption,  
Up to 30 days for heavy consumption.

### PRINCIPLE

Urine toxicity testing relies on simple immunological techniques. The speed and sensitivity of these immunoassays have made them a widely used urine screening method. The sensitivity of this test makes it possible to detect THC and its metabolites as urinary evidence of cannabis consumption.

### IMPORTANT

EXACTO® CANNABIS tests only provide a preliminary analytical result. An alternative chemical method should be used to confirm the result.

### KIT CONTENTS


- 1 pouch containing:
  - 1 Cartridge
  - 1 Pipette
- 1 instruction leaflet
- 1 collection cup

### PRECAUTIONS TO BE TAKEN

- Please read all the information in this leaflet before performing the test.
- Do not use after the expiry date indicated in the aluminium pouch.
  - Keep in a dry place at temperatures between 2 and 30°C. Do not freeze.
  - Do not use the test if the pouch has been torn or damaged.
  - Keep out of reach of children.
  - For external use only.
  - Only open the test pouch when you are ready to use it.
  - Used tests must be discarded adhering to the regulations in force.

**The result must be read after 5 minutes and not interpreted after 10 minutes.**


### INSTRUCTIONS FOR USE

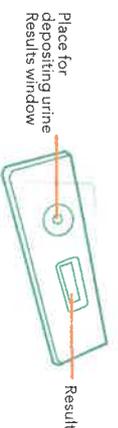- 1** Urinate in the container provided. The urine may be collected at any time of the day.  
Caution: cloudy urine may falsify the test.

- 2** Take a sample of urine using the pipette.

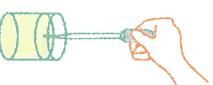- 3** Transfer 3 drops on the test cartridge. After opening, the test must be used within the hour.

- 4** Read the result after 5 minutes. Do not interpret after 10 minutes.

**5 MN**


### RÉSULTAT

**POSITIVE TEST :** C


A coloured band appears in the control area (C). No band appears in the test area (T). This means that THC or one of its metabolites is present at a concentration equal to or higher than the detection limits for the test.

**NEGATIVE TEST :** C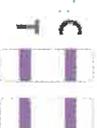

Two coloured bands appear, one in the Control area (C) and one in the Test area (T). The fact that one band is lighter than the other has no impact on the results. This means that THC or one of its metabolites is absent or present at a concentration lower than the test detection limits. The test detection limits are specified below.

**INVALID TEST :** C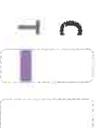

The result is not valid when no coloured band appears in the control area (C), even if a band appears in the test area (T). Repeat the procedure with a new test. (Refer to the instructions for use. Contact us if necessary on +33 (0)3 88 78 85 24)

### DETECTION LIMITS IN URINE :

Substances	Concentration ng/ml
11-nor- D9-THC-9 COOH (THC)	50
Cannabiol	20 000
11-nor- D8-THC-9 COOH	30
D8-THC	15 000
D9-THC	15 000

### INDEX DES SYMBOLES

	Caution, consult the Directions for Use.		Do not re-use.
	Number of tests per box		Use before
	Store between 2°C and 30°C		Batch Number
	Product No.		

RFU\_858836\_ML\_V02201803R03  
REF : 858836

**SERVICE CONSOMMATEURS**  
CONSUMER DEPARTMENT  
+33 (0) 3 88 78 85 24

**BIO SYNEX SA**  
22 Boulevard Sébastien Pant  
67400 ILLKIRCH-GRÄFFENSTADEN  
France  
client@biosynex.com  
www.biosynex.com - www.exacto.fr